



# CAG

Center for Animal Genetics

**Pferdehaar hier aufkleben**

Tape horse's hair here

**Informationen zur Probe**

**Sample information**

Tiername

Animal name

---

Name des Besitzers

Owner name

---

Name des Einsenders

Sender name

---

Entnahmedatum

Date sampled

---



# CAG

Center for Animal Genetics

## Anleitung

Reißen Sie 30 bis 40 Mähnen- oder Schweifhaare aus. Vergewissern Sie sich, dass an jedem Haar ein **Haarwurzeln vorhanden** ist. **Wichtig: Die Haare müssen herausgezogen werden und dürfen nicht abgeschnitten werden.**

Kleben Sie die Haare auf der Rückseite dieser Karte. **Achten Sie bitte darauf, dass der Klebestreifen keinesfalls die Haarwurzeln berührt.**

Zum Schutz der Probe sollte sie in einem **separaten verschließbaren Beutel oder Umschlag verpackt** werden.

## Instructions

Pull 30-40 hairs from the horse's mane or tail. The **follicle (hair root) must be attached**. **Important: Do not cut the hair.**

Tape the hair in this box, making sure that the **tape does not touch the hair roots**.

Place this card with the **hair sample into the plastic bag**. Each sample must be in its own plastic bag.



CAG GmbH - Center for Animal Genetics  
Paul-Ehrlich-Str. 23 | D-72076 Tübingen | Germany

Tel: +49 7071 565 44 850 | Fax: +49 7071 565 44 56  
www.pferde-genetik.de | info@pferde-genetik.de